

Application instructions



1. Open outer packaging.



2. Remove shower pouch.



3. Hold pouch with tape to the top of pouch. Insert catheter into the pouch opening.



4. Remove lower strip.



5. Remove upper strip.




6. Stick firmly to the skin around catheter dressing. To help form a good seal shave or clip hair.

(if present and possible to do so)

to site where pouch adhesive will be applied,

ensure area is clean and dry, and free from creams or moisturisers prior to application.

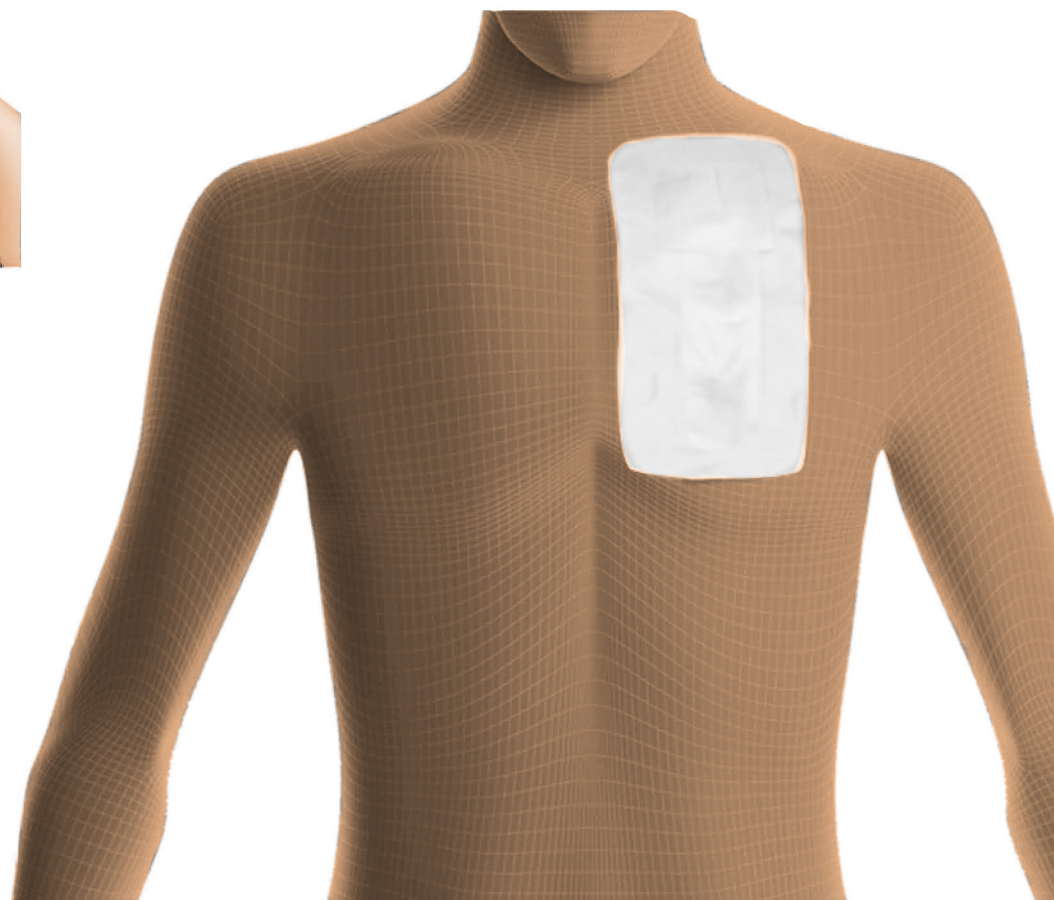

submed Safe Easy

Su Biyomedikal Sistemler ve Sağlık. Hiz. San. ve Tic. A.Ş.
Orhangazi Mah. 1673 Sok. No:20 Esenyurt/ İstanbul/ Turkey
T: +90 212 320 37 53 F: +90 212 320 53 51 info@submed.com.tr
www.submed.com.tr



Distributor : LINC Medical Systems Ltd.
LINC Medical Systems Ltd 7 Kingsley Street Leicester LE2 6DY
T: +44 (0) 1572 717515 sales@linc-medical.co.uk
www.linc-medical.co.uk



INSTRUCTIONS FOR USE

Product Name : Catheter Shower Pouch

Reference No : ND-001

How to Use

Preparation;

1. Open outer packaging.
2. Remove shower pouch carefully, and observe aseptic non touch technique to inner pouch catheter placement area.

Application;

1. Hold pouch with tape to the top of pouch.
Insert catheter into the pouch opening.
2. Remove lower strip.
3. Remove upper strip.
4. Stick firmly to the skin around catheter and catheter dressing (if insitu). To help form a good seal shave or clip hair, and remove creams and moisturisers. Ensure no part of the adhesive dressing is stuck to the catheter dressing or line itself.

Removal

5. Once showering is complete gently peel away shower pouch from skin and around catheter, and dispose of.

Symbols

	Date of Manufacture
	Expiry Date
	Reference Number
	Lot Number
	Do not reuse
	Do not use if package is damaged
	Manufacturer Information
	CE mark
	Medical Device
	Nonsterile

Intended Use

Catheter shower pouches are used to allow the patient to shower protecting the line and the patient from environmental contaminants whilst showering.

User and Patient Information

For adult or pediatric use. Products are to be used by appropriately trained personnel in the healthcare or home setting, and by following these instructions for use.

Storage

Store the product in a cool and dry place protected from sun light and keep dust free.

Disposal

Products may contain fabric, plastic, metallic components This product is considered as medical waste and should be treated according to national and regional laws for the country of use.

Warnings

Product should only be used by suitable trained personnel. Check the integrity of the package before use. If the package is open or damaged do not use. Product is for single use only. Maximum length of application is for period of the shower.

Contraindications

Use only under the guidance of the clinician, if the line itself is damaged/broken, or dressing to the line itself is not intact, or if the area is red or inflamed to around the line seek medical advice before using this product.

Side Effect

If the pouch adhesive area where stuck to the skin, becomes red and inflamed discontinue use immediately and seek medical advice, showing this IFU to the prescribing clinician.